


customerserv@polyflexpro.com

Andrea Edelstein: p. (734) 458-4194 x257

FACE SHIELD

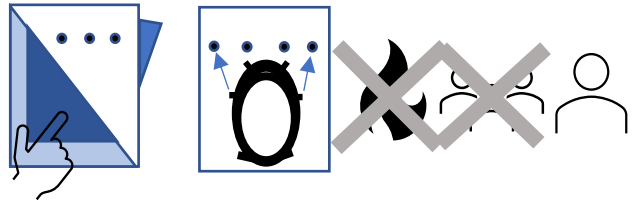

PRODUCT IDENTIFICATION: POLYFLEX text

Materials: HDPE headband, 0.02 PETG and 0.007 Clear Polyester

USE: Face shields are intended for single use, or multiple uses by the same user only. Disinfect between uses.

UNDUE RISK: Use of face shield alone will not prevent infection from microbes or viruses. Not effective against radiation. Do not use in environments where user would have a high risk of contact with chemicals, liquids, or bodily fluids. Not for use in the presence of high intensity heat source, heat sterilization, or flammable gas.

ASSEMBLY: Remove Protective film from **BOTH SIDES** of clear plastic shield prior to use. Insert four pegs on head band into four holes in clear shield. Adjust back strap so the unit fits securely on head.

DISASSEMBLE PRIOR TO CLEANING: If VISIBLY DIRTY-Clean with soap/detergent and water. THEN, disinfect with 70% Isopropyl alcohol, 5% bleach solution, or 28% Hydrogen peroxide. Please follow current CDC guidelines. CDC recommended letting the bleach solution contact the surface for at least 1 minute with 10 min dry time. For other disinfectants, follow the manufacturer's guidelines.

EMERGENCY PREPAREDNESS ACT AND CLAIM TO LEGAL IMMUNITY

The Public Readiness and Emergency Preparedness Act (public Law 109-148 and codified at 42 U.S.C 247d-6d and 42 U.S.C 247d-6e) allows the Secretary of Health and Human Services to issue a declaration to provide liability immunity to certain individuals and entities against any claim of loss caused by, arising out of, relating to, or resulting from the manufacture, distribution, administration, or use of medical countermeasures.

On February 4, 2020 the Secretary of Health and Human Services issued a declaration to provide liability immunity for activities related to medical countermeasures against COVID-19 (the "Declaration")

Polyflex Products, Inc and their independent contractors has manufactured and sold or donated products to you based on the Declaration and resulting immunity protection. The delivery and your acceptance of the products constitutes acknowledgement of this immunity and Polyflex Products, Inc and their independent contractors will rely on such acknowledgement.

THIS FACE SHIELD IS NOT FDA APPROVED FDA: Manufacturers of Face Shields; Health Care Personnel; Hospital Purchasing Departments and Distributors; and, Any Other Stakeholders
<https://www.fda.gov/media/136842/download> This product is authorized for use under the FDA EUA during the Coronavirus Disease 2019 (COVID-19) pandemic, effective April 9, 2020. Authorized under Emergency Use Act (EUA) to be used in medical facilities if an FDA approved face shield is NOT available. All non-FDA approved face shields must be removed from medical use after the EUA is revoked.

Instructions are available online by visiting <https://www.polyflexpro.com/faceshieldinstructions.pdf>